

ARTEX[®] TRANSFER STAND AND TABLE

INSTRUCTIONS FOR USE

The transfer stand (#48670) comes with one transfer table (#46240), and one allen wrench that is stored in the rubber bushing on the top arm of the stand.

The transfer table (also available as a separate item), is vertically adjustable by means of the allen wrench.

TRANSFERRING THE ROTOFIX[®] FACEBOW REGISTRATION TO THE TABLE

1. Remove the 3-D universal joint from the facebow by loosening the attaching screw. (Make sure the registration was taken with the toggle lever to the doctor's right, or else the 3-D joint will not fit into the transfer stand).
2. With the transfer table attached to the stand, loosen the adjustment screw on the table with the allen wrench, and lower the table to its lowest position. Make sure that the threads of the retention screws are protruding through the top of the table.
3. Attach the universal joint to the stand by guiding the threads of the attaching screw into the milled groove on the outside of the stand. At this point, determine if the transfer table is within approximately 1/2" from the bottom of the bitefork. If not, loosen the adjustment screw with the allen wrench, and carefully raise the table. **You may prefer to remove the joint temporarily while accomplishing this adjustment, to avoid the possibility of bumping the bitefork. Then replace the joint one more time to check the table height.**
4. Remove the 3-D joint by opening the attaching screw completely and tipping the bitefork up slightly. Carefully set the 3-D joint aside. Mix an appropriate amount of quick-setting, low-expanding plaster to fill the gap between the top of the table and the bitefork, and place it on top of the table.
5. Replace the 3-D joint onto the transfer stand, beginning with the bitefork tipped slightly upwards as you guide the threads of the attaching screw into the milled groove of the stand. Hold the toggle lever out of the way with your little finger. As you tighten the attaching screw, the bitefork will settle easily into the quick-setting plaster.
6. After the plaster has set completely, open the collar screw at the bitefork. Open the attaching screw and remove the 3-D joint from the transfer stand. The facebow registration is now securely fixed on the transfer table and ready for transport to the laboratory.

