

## PLASTIC BITEFORKS

Jensen plastic biteforks with compound offer time savings, cost savings, and convenience since there is no need to stock or apply thermoplastic bite compound in the operatory. The plastic forks can be disposed of at the laboratory, so there is no need for you to keep track of whether the forks have been returned to the dental office as with metal forks.

**The forks are individually wrapped, but have not been sterilized.**

### USE INSTRUCTIONS

**Sterilize as per your normal cold sterilization procedure.**

Each plastic fork is supplied with thermoplastic bite compound adhered to it. The three points of compound are located in the appropriate positions to provide an incisal edge impression and two posterior cusp-tip impressions.

**Place the fork in a water bath heated to 113°F (45°C) for 3 minutes before its use.**

You may prefer to advise your laboratory that there is no need to return a plastic bitefork to you, and that they can dispose of the fork once the case has been mounted.

Please contact your Jensen Account Manager with any questions, at 800-243-2000.

